

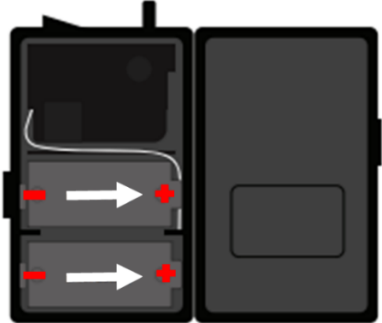



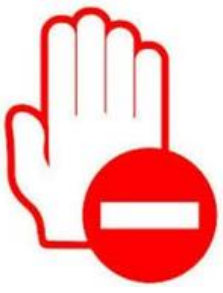




# Biozoon AIR with Flavour® - NHS User Guidelines

 <p><b>Battery Safety</b></p> 	 <p><b>IMPORTANT</b> Insert batteries correctly (+/-) Use only with AIR with Flavour® Do not use to dispense medication</p> <h3>Safe battery Install</h3> <ul style="list-style-type: none"><li>• Ensure batteries are inserted in the correct orientation +/- (as shown in the diagram)</li><li>• Only use good quality alkaline batteries.</li><li>• Do not mix new and old, or different brand batteries.</li><li>• Do not use rechargeable batteries</li><li>• Remove batteries from device when in storage for extended periods of time.</li><li>• Free labels are available, please email <a href="mailto:info@biozoon.uk">info@biozoon.uk</a> to request.</li></ul>
	<h3>What is AIR with Flavour®?</h3> <ul style="list-style-type: none"><li>• An innovative product that can be used to produce aerated foam bubbles from a liquid or drink which dissolves in the mouth to almost nothing allowing an individual to experience “taste for pleasure”</li><li>• This provides comfort and improves quality of life helping to minimise the psychosocial impact of being nil by mouth.</li><li>• For patients with severe dysphagia this presents a significantly lower risk of respiratory aspiration than food or fluid</li><li>• The pump device is not classed as a medical device and is used solely for the preparation of edible foam.</li></ul>
	<h3>Who can use it?</h3> <ul style="list-style-type: none"><li>• Persons who are nil by mouth or have limited oral intake ability.</li><li>• NB. For those with oropharyngeal dysphagia concerns, a Speech and Language Therapist can advise on whether AIR with Flavour® would be suitable.</li><li>• Recommended use along with an individual patient care plan including consideration of: frequency, amount, posture, saliva management, oral hygiene and other specific requirements.</li></ul>

# Biozoon AIR with Flavour® - NHS User Guidelines

 A clipboard with a checklist showing three blue checkmarks and one red 'X'. A red warning triangle with an exclamation mark is overlaid on the bottom right of the clipboard.	<h3>Considerations</h3> <ul style="list-style-type: none"><li>• Maintain good oral hygiene before, during and after use.</li><li>• Ensure that only the foam is given, not the fluid underneath.</li><li>• Can increase saliva production. This increase in saliva could pose an increased aspiration risk in some patients.</li><li>• Having yankauer suctioning available for those who regularly require suction or pose an increased aspiration risk.</li></ul>
 A red outline of a hand with a red circle and a white horizontal bar (prohibition sign) overlaid on the palm.	<h3>Contra-indications</h3> <ul style="list-style-type: none"><li>• Not recommended those at risk of respiratory deterioration or known respiratory infections, saliva management issues and/or poor oral hygiene.</li><li>• Do not continue use if repeated coughing, hypersalivation or wet voice is evident.</li><li>• Soya, Soya oil or Maltodextrin allergies</li></ul>
 A hand holding a black sign with the word "ALLERGY" written in white capital letters.	<h3>Ingredients &amp; Allergy</h3> <ul style="list-style-type: none"><li>• There are two food grade ingredients which are Maltodextrin and Soy Lecithin, these help to emulsify the liquid and stabilise the foam.</li><li>• Check allergies with patient prior to use, If patient unable to express this then check with NOK/family and medical documentation.</li><li>• Ensure person administering is not allergic to soya due to risk of aerosolization</li></ul>
 A silver sachet with the text "AIR AIR AIR instant" printed on it.	<h3>Sachets</h3> <ul style="list-style-type: none"><li>• A new sachet is required for each use</li><li>• Sachets are single use and should be disposed of after use.</li><li>• Check "Best Before Date" on main multipack before use and discard if out of date.</li><li>• Do not use sachet if damaged or open</li></ul>

# Biozoon AIR with Flavour® - NHS User Guidelines



## Operation & Use

- Only use the products as intended. Do not use with any other product or medication.
- Only to be used by staff or patients who are instructed in the correct use and operation of the product.
- Always prepare the mixture in a clean and suitable location.
- Ensure pump device and silicone tubing are clean and in good condition before use.
- Do not allow silicone tubing to contact liquid or foam. Only the pipette contacts liquid.
- Replace silicone tubing if visibly damaged, soiled, discoloured or had contact with liquid.
- Silicone tubing is single patient use.
- Aeration pipettes are single use.
- Do not allow the pump device to become immersed in liquid.



## Cleaning & Infection Control

- Ensure device and tubing are clean and in good condition before and after use.
- Pump device is cleaned with a detergent or 70% isopropyl alcohol wipe.
- Care should be taken when cleaning not to wet excessively or immerse in any liquid.
- Silicon tubing is cleaned externally with a detergent or 70% isopropyl alcohol wipe. The inside is not cleaned, it only transports ambient air
- Do not allow tubing to contact liquid or foam. If contamination occurs, replace and dispose of soiled tubing.
- Silicone tubing is single patient use, replace if visibly damaged, soiled or discoloured and/or after 30 days use.
- Pipettes are single use and are disposed of after each use.
- For blood and body fluid contamination clean pump unit using a 10,000 ppm solution of chlorine. Dispose of silicone tubing and pipette.
- Check with your in-house IPC for local infection control policy.